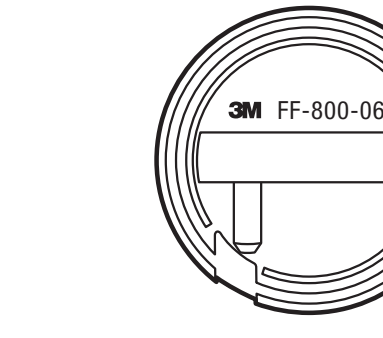
Secure Click™ Quantitative Fit Test Adapter FF-800-06

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IMPORTANT: before use, the wear must read the *User Instructions* and the *User Instructions* for the respirator facepiece. Keep these instructions for reference.

⚠ WARNING

This respirator helps protect against certain airborne contaminants. **Misuse may result in sickness or death.** For correct use, consult supervisor and *User Instructions*, or call 3M in U.S.A. at 1-800-243-4630. In Canada, call Technical Service at 1-800-267-4414. In Brazil, contact 0800-0132333.

Description

The 3M™ Secure Click™ Quantitative Fit Test Adapter FF-800 Series is designed to permit quantitative fit testing of the 3M™ Secure Click™ FF-800 Series Full Facepiece Reusable Respirator (FF-801, FF-802, and FF-803).

Adapter Installation Instructions:

- Remove exhalation valve cover by gently pulling left and right side of cover away from the facepiece (Fig. 1).
- Remove the nose cup assembly by grasping the hard plastic ring of the nose cup where it attaches to the lens and gently pulling it up and away from lens assembly (Fig. 2).
- Remove exhalation valve assembly by turning counter clockwise 1/4 turn (Fig. 4).
- Remove the speaking diaphragm by turning counter clockwise 1/4 turn (Fig. 4).
- To install the fit test adapter, align it with the speaking diaphragm opening so that the fit test port is facing right (Fig. 5).
- With slight downward force, use the thumb and forefinger to rotate the adapter ½ turn clockwise, so that the fit test port is facing downward (Fig. 6).
- Visually inspect the fit test adapter for misalignment or incomplete placement. If there's any indication of incomplete placement, remove the fit test adapter and repeat the installation procedure.
- Attach the tubing to the fit test port. Pull gently to ensure it is secure (Fig. 7). NOTE: Tubing is not included with the FF-800-06.
- Reattach the exhalation valve assembly, being careful not to pinch the attached tubing (Fig. 8).
- Replace the nose cup by aligning hard plastic ring with lens assembly and pressing firmly on center tabs until bottom of the nose cups snap in place. Press down on top of nose cup ring until tabs snap in place.
- Reattach the exhalation valve cover, being careful to ensure the tubing is coming out the opening at the bottom of the cover and is not pinched (Fig. 9 and 10).

Ensure the appropriate filters are installed prior to beginning the fit test.

The fit test adapter must be removed and replaced with the speaking diaphragm prior to using the facepiece in the work environment. Speaking diaphragm installation instructions are located in the FF-800 Facepiece Product User Instructions.

⚠ WARNING

The quantitative fit test adapter must be removed and replaced with the speaking diaphragm prior to using the respirator in a contaminated environment. Failure to follow these instructions may reduce respirator performance, expose you to contaminants above the OEL, and **may result in sickness or death.**

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⚠ WARNING

The Adapter for the quantitative Dichtungsprüfung must be removed and replaced with the speaking diaphragm before use, because the Dichtungsprüfung is in a contaminated environment. Failure to follow these instructions may reduce the performance of the Dichtungsprüfung and expose you to contaminants above the OEL, which can **lead to illness or even death.**

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