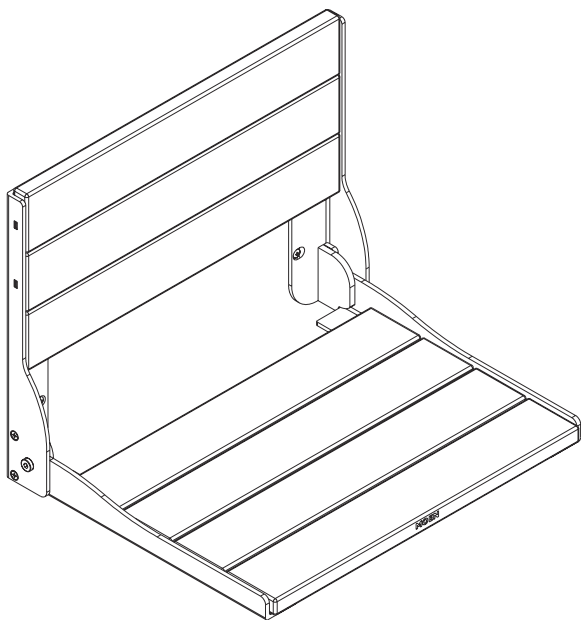
Instructions for the Wallmount Teak Shower Chair

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Components Included

- Teak Shower Chair
- #10 Wood Mounting Screws - Qty 6

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Tools Required

- Pencil
- Level
- Stud Finder
- Drill with Phillips bit #2
- Protective eyewear

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Safety Information

- For safe and secure installation, ensure the seat is mounted into wood studs/backing
- The shower seat should not be installed into fiberglass/acrylic shower enclosures
- Shower seat should be installed by a qualified professional
- Seat should only be used for sitting and for no other purposes
- Seat is limited to one person at a time with a maximum weight capacity of 400 lbs

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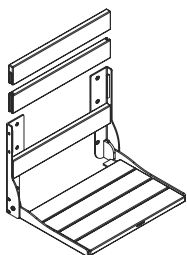
Warning ! – FOLLOW THESE INSTALLATION INSTRUCTIONS CAREFULLY. PROPER INSTALLATION IS THE INSTALLERS RESPONSIBILITY. FAILURE TO FOLLOW CORRECT INSTALLATION PROCEDURES CAN CAUSE THE SEAT TO BECOME LOOSE, WHICH CAN RESULT IN SERIOUS INJURY.

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2

2. Identify the mounting location of the shower seat making sure the shower seat is level.

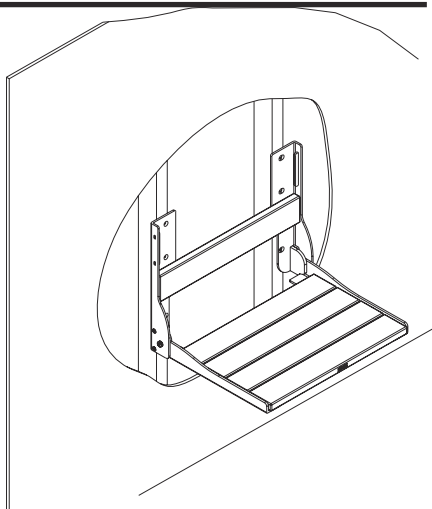
- a. *Note: Be sure the mounting holes are located over studs or wood backing.*
- b. *Note: Recommended seat height is 15" – 21" from the shower floor.*

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
3

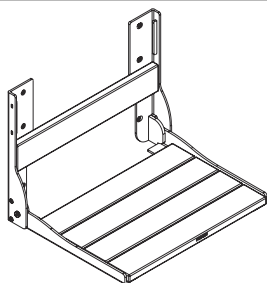3. Mark the 6 mounting hole locations once the desired location is determined.

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4

4. Set aside shower seat and drill 1/8" pilot hole through tile in the each mounting location. Drill a 1/4" hole thru the tile only at these same locations to allow the wood screw to pass thru.

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5

5. When complete, ensure the wall is clear of any debris.

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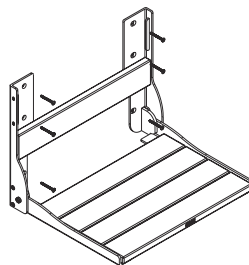
6


6. Secure the shower seat to the wall, using the provided 6 wood mounting screws. Start with the top two mounting hole locations.

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7

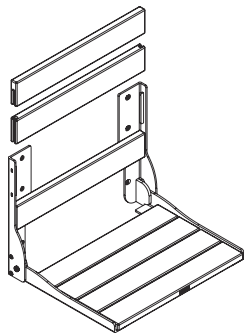7. Once all screws have been installed and tightened, slide the back panels into place.

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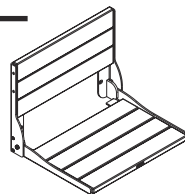
8

8. Installation is complete – Test the seat to make sure it is secure.

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Cleaning and Teak Wood Care

- Clean the seat with a warm, damp cloth
- The natural properties of Teak allow for the wood to weather over time when exposed to the environment. If you desire to keep the current look of the wood, treat the teak with a certified Teak oil.

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WARNING:

TO PREVENT SERIOUS OR FATAL INJURY: FOR USE ONLY BY PERSONS WEIGHING LESS THAN 400 POUNDS. Consult a healthcare professional for advice and instructions specific to your needs before use. Contact Moen immediately at 1-800-882-0116 if any components are missing or damaged. Install in accordance with Moen's instructions. Do not modify this product in any way. Check grab bar periodically after installation to be sure all screws are secure. FOR QUESTIONS OR ASSISTANCE WITH INSTALLATION CONTACT MOEN AT 1-800-882-0116.

LIMITED LIFETIME WARRANTY:

If this product fails due to materials or workmanship at any time during the life of the product Moen will replace it free of charge, postage-paid. Simply contact Moen at 1-800-882-0116 for replacement information. This warranty does not cover products which have been abused, altered, damaged, misused, cut or worn. TO THE EXTENT PERMITTED BY LAW, MOEN DISCLAIMS ALL OTHER IMPLIED OR EXPRESS WARRANTIES INCLUDING ALL WARRANTIES OF MERCHANTABILITY AND/OR FITNESS FOR A PARTICULAR PURPOSE.

LIMITATION OF LIABILITY: THIS WARRANTY IS YOUR SOLE REMEDY AND MOEN SHALL NOT BE LIABLE FOR ANY DAMAGES, WHETHER DIRECT, INDIRECT, INCIDENTAL, SPECIAL, CONSEQUENTIAL, EXEMPLARY, OR OTHERWISE, INCLUDING LOST REVENUES AND LOST PROFITS, ARISING OUT OF ANY THEORY OR RECOVERY, INCLUDING STATUTORY, CONTRACT OR TORT. NOTWITHSTANDING THE TERM OF ANY LIMITED OR IMPLIED WARRANTY, OR IN THE EVENT THAT ANY LIMITED WARRANTY FAILS OF ITS ESSENTIAL PURPOSE, IN NO EVENT WILL MOEN'S ENTIRE LIABILITY EXCEED THE PURCHASE PRICE OF THIS PRODUCT. SOME STATES AND PROVINCES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY TO YOU. THIS LIMITED WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE RIGHTS WHICH VARY FROM STATE TO STATE AND PROVINCE TO PROVINCE.

FOR ADDITIONAL WARRANTY INFORMATION, VISIT WWW.MOEN.COM
MEETS OR EXCEEDS APPLICABLE REQUIREMENTS FOR THE AMERICANS WITH DISABILITIES ACT.

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